

INSTRUCTIONS FOR

DELUXE TUBE INSERTER (WWW.Hearing-Loss-Help-Co.com)

Please read these simple instructions carefully to insure best results.

1. Pull the wire loop (B) out of the TUBE INSERTER handle (A) until it is fully extended. Slide the small brass bead (C) back on the wire loop, toward the handle, as far as it will go.
2. Put the end of the wire loop in the canal opening (D) of the ear-mold and work the loop carefully through the earmold until it projects from the opposite opening.
3. Take the plastic tubing (E) to be inserted in the earmold and push the end of it about 1/8" through the projecting wire loop from the outside of the curved portion, as shown.
4. Holding the earmold securely in one hand and the handle of the TUBE INSERTER in the other, pull steadily on the TUBE INSERTER so as to draw the tubing completely through the earmold.

IMPORTANT: Pull in a straight line with the canal opening of the earmold.

5. Carefully draw the tubing back into the earmold as far as desired. Cement, if used, should be applied to the tubing first, but since the tubing runs entirely through the earmold, cement is not usually necessary.
6. The above instructions apply to the insertion of straight or preformed tubing in new earmolds, or to the replacement of such tubing in earmolds from which the old tubing has been completely removed.

